

ProConnect code: "tryonmed"

Linking Omnipod 5 to Glooko

Step 1: If your Omnipod 5 device was not linked to Glooko when you first registered your device, it must be done in Podder Central before your Healthcare Professional can be connected to Glooko. Complete the below in your browser (Safari, Chrome, Firefox, Explorer - not the app) to select the "Diabetes Data" tab in your Podder Central account to link your pump data.

Link your Omnipod® 5 Automated Insulin Delivery System data with Glooko®.

Follow these four simple steps:

- 1 Log into your Omnipod® account at [PodderCentral.com](https://poddercentral.com)
- 2 Select "**Diabetes Data**" and then select "**Link**"
- 3 Select "**Create or Link Account**" in Glooko®
- 4 Proceed to Glooko® and complete the process to link your accounts

When complete, your status will update to "**Linked**" and data will automatically upload into your personal Glooko® account.*



Pod and Dexcom G6 CGM shown without necessary adhesive.



Questions? Contact Insulet Product Support at [1-800-591-3455](tel:1-800-591-3455)

Insulet Corporation 100 Nagog Park, Acton, MA 01720 * omnipod.com

*Must be connected to WiFi or cellular data, and Omnipod® 5 users must have WiFi or cellular data access to receive updates in Glooko®.
© 2022 Insulet Corporation. Omnipod and the Omnipod logo are trademarks or registered trademarks of Insulet Corporation. All rights reserved. Glooko is a trademark of Glooko, Inc. and used with permission. Dexcom is a registered trademark of Dexcom, Inc. and used with permission. All other trademarks are the property of their respective owners. The use of third party trademarks does not constitute an endorsement or imply a relationship or other affiliation. NC-045-01-2022-00028 V1.0

Glooko account active and linked in Podder Central: If you linked your pump data to your healthcare professional during device registration in Podder Central and have a Glooko account (or completed step 1), you can add the above ProConnect code directly by following the steps below to link your data to your specific Healthcare Professional.

Share your Omnipod® 5 Automated Insulin Delivery System data with your healthcare provider.

Entering a **ProConnect code** in Glooko® is the only way for your healthcare provider to see your Omnipod® 5 data at your next visit.



Five simple steps:

- 1 Log into Glooko® at my.glooko.com
- 2 Select "**Settings**" from the drop-down menu (top right on the home screen)
- 3 Scroll down to **Account** and select "+ Add New Code"
- 4 Select "**Continue**"
- 5 Enter ProConnect Code (listed below) then select "**Submit**"

ProConnect Code: _____



Please confirm your pump data is linked by signing into your Glooko account directly at my.glooko.com. On the upper right side of the home screen next to your initials, select "Settings". Scroll down to "Account" and confirm your Healthcare professional's clinic is listed under "Primary Provider".

